

ReminGel®

Dental Desensitizing / Remineralizing Agent

Remineralizing Mint

PRODUCT DESCRIPTION:

ReminGel is a convenient, premixed, ready-to-use, opaque aqueous-based gel. ReminGel consists of suspended hydroxyapatite and calcium and phosphate salts that help restore a tooth's hydroxyapatite structure through remineralization. This remineralization results in tubule occlusion and blocks fluid flow in dentinal tubules, thereby relieving dentinal sensitivity. Additionally, ReminGel blocks nerve excitability through potassium release.

INDICATIONS FOR USE:

- To relieve dental hypersensitivity caused by bleaching procedures, cold, heat, acids, or sweets.

INTENDED USERS:

- Dental professionals may prescribe ReminGel to patients for their use.
- Patients may directly purchase ReminGel for use.

CONTRAINDICATIONS:

- Do not use ReminGel in patients with a known allergy to any of the product's ingredients. An allergic reaction may require treatment.

PRECAUTIONS:

- Consult accompanying Safety Data Sheet (SDS).
- Do not use after the expiration date.
- ReminGel is not sterile and is not intended to be sterilized. Failure to follow these instructions could damage the product resulting in procedural delays or user inconvenience.

STORAGE:

- Store at room temperature between 18°C (64°F) and 30°C (86°F) in a cool, dry place.

WARNINGS:

- Keep out of reach of children.
- For dental use only.
- Always check the expiration date prior to use.
- Do not use if package is damaged.
- Stop use and ask a doctor if:
 - 1) sore mouth symptoms appear and do not improve within seven days,
 - 2) irritation, pain or redness persists or worsens, or
 - 3) swelling, rash, or fever develops.
- ReminGel has not been tested on pregnant or nursing mothers.





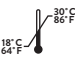






DIRECTIONS FOR USE

(Application using a Dental Appliance such as a Fluoride Tray, Whitening Tray, PerioTray, etc.):


1. Cut tube tip close to the narrow end.
2. Express a ribbon of ReminGel in the dental appliance tray.
3. Insert tray containing ReminGel into the mouth in typical fashion such that the ReminGel contacts the teeth.
4. Spit out any excess ReminGel.
5. Leave ReminGel in contact with the teeth for 3-15 minutes. Recommended contact duration: 15 minutes, unless otherwise directed by your dentist or doctor.
6. Remove tray and rinse mouth with water. A toothbrush may be helpful to remove any residual ReminGel from the teeth surface.
7. Use up to four times daily or as directly by a dentist or doctor.

SYMBOL GLOSSARY:

	Manufacturer
	Consult instructions for use
	Keep dry
	Do not use if package is damaged
	Temperature limitations for storage
	Caution – consult accompanying documentation
	Batch number (shown as LOT on tube crimp)
	Expiration date (shown as EXP on tube crimp)
	Non-Sterile

REF RGEL-3000

Manufactured for:
PerioProtect, LLC
9701 Green Park Industrial Dr.
St. Louis, MO 63123 USA

 Inter-Med Inc.
2200 South St.
Racine, WI 53404 USA
Tel: +1-262-636-9755

MADE IN USA

3002-I-V-ENG (1)

