

Instructions for Dexis IS Scanners

Dexis IS

Invite Perio Protect Lab to be a laboratory partner by following these steps.

- Log in to Dexis IS as a dentist
- In the tool bar select this icon: 
- In the Partners window, click "INVITE A PARTNER"
- In the email field enter: 3d@perio-protect.com
- Click [SEND](#)

Once we accept, you may start sending cases!

PerioProtect Laboratory
9701 Green Park Industrial Drive
St. Louis, MO 63123

Phone: 314-474-0467
Fax: 888-511-2308
Email: lab@perio-protect.com
Web: perioprotect.com/3D